

Figure 1

Pride wheelchairs have a different method for disengaging and re-engaging their motors. Unlike other power wheelchairs, models like the Jazzy have their switches located at the front of the frame, right behind the footplate (**Figure 1**). There are two of them, which are side-by-side, and are yellow in color.

Figure 2

Grab the switch on your left and press it down (**Figure 2**). You will feel a slight resistance as you press the switch down.

Figure 3

After you have pressed the switch down completely (**Figure 3**), you'll feel and hear a soft pop. When this happens, you have successfully disengaged the corresponding motor.

Figure 4

Repeat this process for the second yellow switch (**Figure 4**). This one is located directly next to the previous switch. Once both switches are in the downward position, the power wheelchair will now be in “Free Wheel” mode, and can now be pushed.

Figure 5

To re-engage the motor on your right, grab the corresponding switch and pull upward (**Figure 5**). You will feel another soft pop as the motor is put back into “Drive” mode.

Figure 6

Repeat this process to re-engage the motor on your left, pulling it into the upward position (**Figure 6**). Once both switches are in the upward position, the power wheelchair will now be in “Drive” mode, and can no longer be pushed.