

If you have a Quickie power wheelchair, and you need to engage and/or disengage its motors, here are the steps. Depending on the model that you have, there will either be a **red switch**, or a **black lever**. (Note: Your wheelchair has two motors.)

Figure 1 (Red Switch)



If your switch is red (**Figure 1**), you can either rotate it clockwise or counterclockwise. As indicated by the sticker on the side of each motor... **(Circled in red)**

Figure 2 (Red Switch)



Rotating the switch upwards (**Figure 2**) will put it in the “Up” position, which will set the motor to “Drive” mode.

(Repeat this process for the other motor)

Figure 3 (Red Switch)



Rotating it downwards (**Figure 3**) will put the switch into the “Down” position, setting the motor to “Free Wheel.”

(Repeat this process for the other motor)

Note: These red handles will only move a little bit in either direction, and that you’ll hear a loud, distinctive click when you change their position.

Figure 4 (Black Lever)

For motors that have the black lever (**Figure 4**), it can either move up or down.

Figure 5 (Black Lever)

Flipping the switch into the “Up” position (**Figure 5**) will set the motor to “Drive” mode...

(Repeat this process for the other motor)

Figure 6 (Black Lever)

...while flipping it to the “Down” position (**Figure 6**) will set the motor to “Free Wheel.”

(Repeat this process for the other motor)

Note: You will feel a slight resistance before the lever pops into the intended position.